
PROZIS

FLOWXY

NASAL DILATOR

User Manual
Warranty Statement

V.1 04/2026

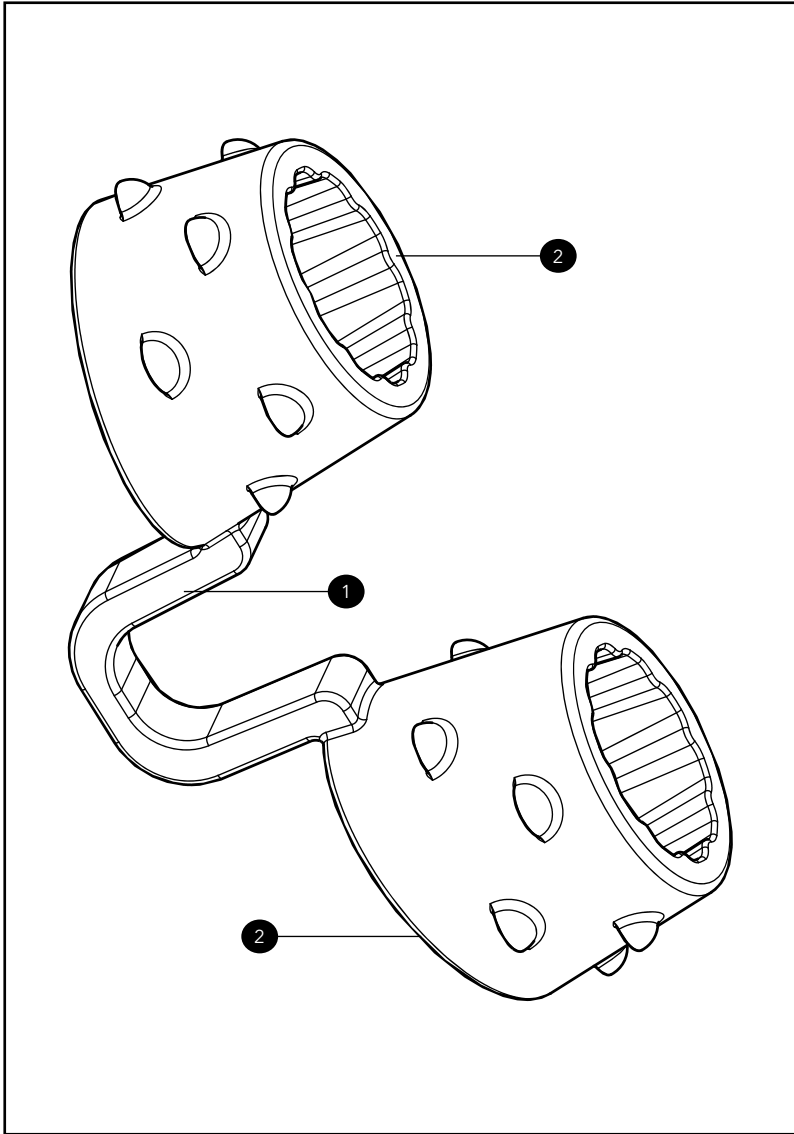Language index

EN	4
FR	10
DE	17
IT	25
ES	32
PT	39

Prozis is a registered trademark of PROZIS.COM, S.A.

Prozis reserves the right to adjust or modify the product or any of the associated documentation in order to ensure its suitability for use, at any time and without prior notice.

NOTE: The most recent version of this product's user manual can be found at www.prozis.com/user-manuals

Customer support and recycling information

Prozis will always be available for any additional explanation through our customer support communication channels, available at www.prozis.com/support

The packaging that protects the device against transport damage is made from non-polluting materials that can be disposed of via local recycling bins. The recycling code is used to identify various materials for recycling. The code consists of the recycling logo, which is meant to reflect the recycling type, and a number that identifies the material.

The product and packaging materials are recyclable. Dispose of them separately for better waste treatment. The Triman logo is valid in France only.

Safety warnings

Get to know the product before starting to use it. Carefully read the operating instructions and warnings. Use the product only as described and only for the given areas of application. Keep these instructions safe. When passing the product on to a third party, always make sure that the documentation is included. The manufacturer is not responsible for any injuries or health complications arising from the improper or unsupervised use of this equipment.

Intended use

This product is designed to support nasal breathing both while sleeping and during physical activity. By gently opening the internal nasal passages, it helps improve airflow, increase breathing comfort, and reduce the sensation of nasal congestion, promoting a more regular and effective breathing pattern.

DANGER! This symbol, when combined with the note **Danger**, means high risk. Failure to observe this warning may lead to injury to life and limb.

DANGER FOR CHILDREN AND PEOPLE WITH DISABILITIES

- This product is not a toy and must not be used by children.
- It is not intended for use by individuals with physical, sensory, or cognitive impairments that may prevent them from understanding or correctly following the instructions for use.

- Improper handling may result in injury from incorrect insertion, excessive force, or accidental ingestion or aspiration.
- To avoid serious injury, keep this product out of the sight and reach of children at all times.
- Persons with disabilities should only use this product under the direct and continuous supervision by a responsible adult.

DANGER! RISK OF INJURY!

- Incorrect use of this device may cause injury to the nasal cavity, irritation, discomfort, or breathing difficulties.
- Do not use this product if you have open wounds, inflammation, infection, or frequent nosebleeds, or if you have recently undergone nasal or sinus surgery.
- Discontinue use immediately if pain, irritation, abnormal pressure, or breathing difficulty occurs.

DANGER TO AND FROM PETS AND LIVESTOCK

- Due to its small size, this product may pose a choking or ingestion hazard to pets and livestock.
- Animals may chew or swallow it, potentially causing serious injury to the animal and contamination or damage to the device.
- Always store the product in a closed container, out of the reach of animals.
- If the product has been bitten, deformed, or contaminated by an animal, do not use it again and replace it immediately.

CAUTION! This symbol highlights dangerous situations that can lead to minor to moderate injuries, damage, malfunction, and/or destruction of the device.

To prevent harm or damage from happening to users or others, make sure to comply with the following requirements.

CAUTION! RISK OF MATERIAL DAMAGE

- Improper handling or care may result in damage, deformation, or premature deterioration of the product, reducing its effectiveness and service life.
- Do not expose the device to extreme temperatures.
- Do not boil, microwave, or clean with harsh chemicals, alcohol, or abrasive agents.
- Avoid excessive stretching, twisting, or mechanical stress beyond normal use.
- Store the product in a clean, dry place away from direct sunlight and heat sources.
- In case of any visible damage such as cracks, tears, loss of elasticity, or deformation, discontinue use immediately.

CAUTION! GENERAL ADVICE

- To prevent any harm to users or others, ensure that the product is inserted gently without excessive force and that it is correctly positioned before engaging in physical activity.
- After each use, clean the device with warm water and mild soap and allow it to dry completely before storing it.

- Never share the product with others. It is strongly advised to replace the product after a maximum of 30 uses, even if no visible damage is present.
- While this device can be used while sleeping, involuntary movement may cause displacement, potentially affecting breathing comfort.

User manual

Package contents

- 2 x Flowxy - Size S
- 2 x Flowxy - Size L
- 1 x Quick-Start Guide

Product overview (see page 2)

1. Central Connector
2. Nasal Tubes

Maintenance and cleaning

Cleaning:

- Clean the device after each use. Always clean it immediately after training or physical activity to remove sweat, skin oils, and residues that may accumulate during use.
- Clean the device with warm water and a mild, neutral soap. Gently rub the surface of the device with your fingers to ensure all areas, including the textured grip areas, are thoroughly cleaned. Rinse carefully with clean water to remove any soap residue.
- After washing, allow the device to air-dry completely in a clean, well-ventilated area. Ensure it is fully dry before storing or reusing it to prevent hygiene issues or material degradation.
- Do not boil, steam-sterilize, microwave, or expose the device to high temperatures. Do not use alcohol, solvents, disinfectants, abrasive cleaners, or chemical agents, as these may damage the material and reduce its elasticity and lifespan.
- Do not place the device in a dishwasher or washing machine. Do not use brushes or sharp objects for cleaning, as these may damage the microtextured surface of the device.

Storage:

- After cleaning and fully drying the device, store it in a clean, dry, and well-ventilated place.
- Proper storage helps maintain hygiene, material integrity, and product performance over time.
- Keep the device away from direct sunlight, heat sources, and extreme temperatures. Prolonged exposure may cause material degradation, deformation, or loss of elasticity.
- Do not store the device near radiators, heaters, or inside vehicles exposed to heat.
- Store the device in its original packaging or in a dedicated protective container to prevent contamination, dust accumulation, and accidental damage. Avoid contact with sharp or heavy objects or materials that could compress or deform the device during storage.
- Do not store the device while wet or damp. Ensure it is completely dry before storing to prevent microbial growth or unpleasant odors.


- Do not store the device in environments with high humidity or exposure to chemicals, solvents, or cleaning agents. If the device shows signs of deterioration, deformation, or material fatigue during storage, discontinue use and replace it.

Instructions

How to use


- Before first-time use, make sure that the device is clean, dry, and undamaged.
- Wash your hands thoroughly before handling the product.
- Pick your size. If you are unsure, try both sizes for best comfort and airflow.

How to insert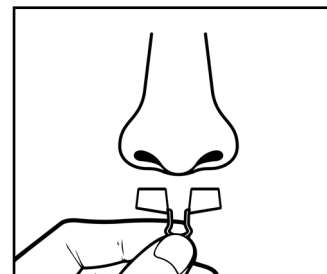

HOLD

Hold the device by the Central Connector. Position the Nasal Tubes just below the entrance of both nostrils.

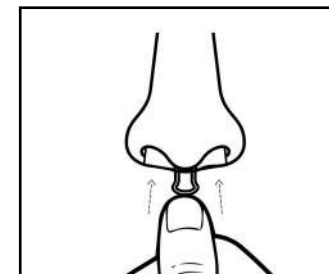
INSERT

Carefully insert the Nasal Tubes into the nostrils. Use slow and gentle movements, following the natural angle.

POSITION

Once inserted, adjust the device's position slightly until it sits comfortably.

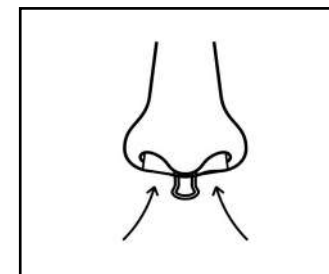BREATHE

When properly positioned, you should experience a subtle, even expansion of the nasal passages. The fit should feel secure and stable without causing pain or discomfort.

TIP: when using the device for physical activity, ensure secure and comfortable fit before exercise.

Removal and Cleaning

- Remove the device by holding the Central Connector and pulling it out in a gentle, controlled motion. Avoid twisting, pulling abruptly, or applying force.
- Wash with warm water and mild soap.
- Allow it to dry completely before storing.

Please refer to the **Maintenance and cleaning** section for further cleaning instructions. Do not reuse the device before it has been cleaned properly. If you feel any pain, irritation, or discomfort during insertion or removal, discontinue use immediately.

Warranty statement

All products commercialized by Prozis via www.prozis.com are covered by the warranty applicable to the purchase and sale of consumer goods.

GENERAL CONSIDERATIONS

This document contains the terms and conditions of Prozis' warranty for final consumers. This warranty is valid exclusively for consumers who purchase the product for non-professional purposes.

CONSUMABLES

The gradual performance decrease caused by prolonged use of the product's consumable components, such as batteries, is not covered by this warranty unless the malfunction is caused by faulty design, materials or production.

HOW TO MAKE A CLAIM

If the need arises to exercise the rights conceded to you through this warranty, please contact Prozis customer service via the communication channels provided on the Prozis website, and follow the provided instructions or recommendations. Contact and technical assistance information is also provided at the end of this warranty.

Any claims made regarding this warranty will only be valid if proof of purchase is presented by the original customer. This proof of purchase may be the original invoice or receipt, as long as it includes information regarding the date of purchase and the product's model name.

WARRANTY REPAIRS

Any repairs made under this warranty cannot be made by third parties. This warranty does not apply to any repairs, or damage caused directly by such repairs, that have impacted or caused the damage that is the subject of the respective warranty claim.

EXCLUSIONS

The following situations are not covered by this warranty:

1. Any wear and tear of parts and components resulting from normal product use.
2. Cracks, dents, scratches, and other types of superficial damage that only affect the product's appearance.
3. Any malfunction caused by inappropriate use of the product that does not comply with user manual instructions.
4. Use and storage of the product not complying with user manual instructions.
5. Opening of the product by a third party who is not authorized to proceed to its repair.
6. Altering or changing parts or components, whether internal or external.
7. Damage caused by battery leakage resulting from the breaking of any of its components, or from product misuse.
8. Malfunction or damage due to reasons that cannot be ascribed to the production or design of the product.
9. Use of the product in the context of a business, occupation or commerce.
10. Malfunction due to use that does not comply with the technical or security norms in force, or with user manual instructions.

The services provided by Prozis in order to repair or fix any fault or malfunction resulting from the verification of any of the aforementioned excluded situations will be subject to payment of labor, transport and component costs.

The customer shall bear all risks of loss and damage to the product during transportation to Prozis. This warranty will be void if the returned product arrives with labels or stickers that have been removed, damaged, tampered with or modified in any way.

LIMITED RESPONSIBILITY

Prozis cannot be held responsible for indirect damage or loss due to usage of the product.

WHAT TO DO?

To request warranty service, you must first contact Prozis customer service via the communication channels provided on the Prozis website, and follow the provided instructions and recommendations.

Send back the product inside its original packaging, including all accessories and documentation, to the following address:

PROZIS
Rua do Cais n° 198
4830-345 Póvoa de Lanhoso
Braga - Portugal

CONTACTS

Prozis is always available for additional clarification through their customer service communication channel at www.prozis.com/support

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PRZ NOW LLC - Sunrise, FL 33325